

SIEMENS



www.usa.siemens.com/hearing

easyPocket™ Quick Setup Guide

For Audiologists and Hearing Care Providers



Life sounds brilliant.

easyPocket™ Quick Setup Guide

Key lock switch

- Slide to lock keys.

Program change button


- Press to change current listening program.
- Press to activate display while in standby. Press again to change program.

Volume up button

- Press to increase volume.

Volume down button

- Press to decrease volume.

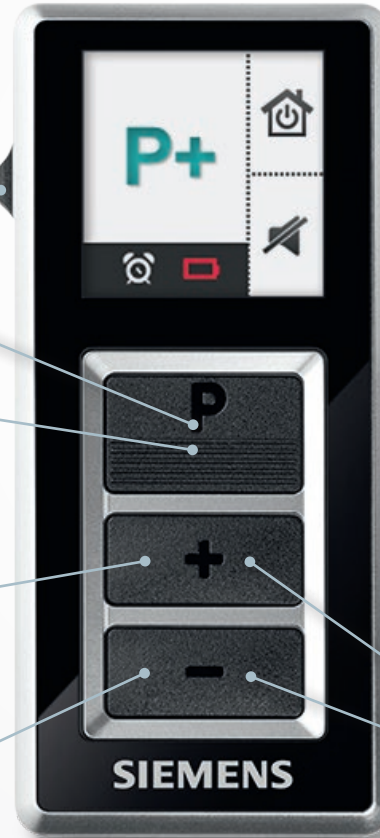

Display

- Indicates change of listening program and volume
- Listening program name
- Listening program icon
- Listening program number

default



if programmed



Display

- Status bar (alarm, easyPocket™ battery)
- Function of control buttons

Upper control button

- Press to reset hearing instruments to default setting (program / volume)

Lower control button

- Press to mute hearing instruments

Volume up + down button (Simultaneously pressed)

- Enter / exit auxiliary menu.



Left side, front, right side



- 1 Select remote control easyPocket.
- 2 Activate easyPocket for the hearing instruments – easyPocket is ready to use with default settings.
- 3 Connect easyPocket by using an USB programming cable to your computer – individualize your easyPocket with programmable settings.
- 4 Check if the program labels are shown correctly in the preview display. If not, then change the program labels in menu “Program Handling”.
- 5 Adjust the configuration of the upper and the lower control button of the easyPocket.
- 6 Adjust the language of the easyPocket auxiliary menu.
- 7 Write your contact data and tick the tick box, if you want them displayed.
- 8 Update easyPocket.

The *Bluetooth*® word mark and logos are registered trademarks owned by Bluetooth® SIG, Inc., and any use of such marks by Siemens AG is under license. Other trademarks and trade names are those of their respective owners.

Hearing instruments help many people hear better, but no hearing instrument can solve every hearing problem nor restore normal hearing.

Information in this brochure is subject to change without notice. Copyright © 2014 Siemens Hearing Instruments, Inc. All rights reserved.

Global Siemens Headquarters

Siemens AG
Wittelsbacherplatz 2
80333 Muenchen
Germany

Global Business Unit

Siemens Audiologische
Technik GmbH
Gebbertstrasse 125
DE-91058 Erlangen
Germany
Phone: +49 9131 308-0

Global Siemens Healthcare Headquarters

Siemens AG
Healthcare Sector
Henkestrasse 127
91052 Erlangen
Germany
Phone: +49 9131 84-0
www.siemens.com/healthcare

Legal Manufacturer & Local Contact

Siemens Hearing Instruments, Inc.
10 Constitution Avenue
Piscataway, NJ 08854-1397
Phone: (800)766-4500
Fax: (732)562-6696
usa.siemens.com/hearing