

Important First Fit Programming

signia

Life sounds brilliant.

The hearing aids have been First Fit
with the Universal Program.



Sivantos, Inc. is a Trademark Licensee of Siemens AG. Copyright © 2016 Signia
GmbH. All rights reserved.

10537367 5/17 5.0 22B771 10K SI/17638-17

Hearing
Systems


SIEMENS

SEE REVERSE SIDE

Important First Fit information for hearing aid orders*



Hearing aids have been first fit. If programming changes are needed, simply click the icon of the BTE with the green arrow in the tool bar to “Detect connecting hearing systems” in CONNEXX®.

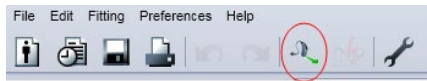

Next, click the radio button “Read out connected instruments” to view the factory-programmed settings in First Fit.

To add additional programs, click “Program Handling” and check off any programs that may be required.

NOTE: Do not click the radio button called “New fitting” or the first fit setting will be cleared. The First Fit Program settings have been optimized to make patient fittings easier and quicker.

*if easyTek™ is ordered the hearing aids have been paired.