

# Charging your Li-ion hearing instruments

- ▶ Connect the inductive charger to the power supply.



One green LED for each charging slot indicates the charger has power and is ready for use.

- ▶ Place the hearing instruments in either charging slot as shown in the picture. Your hearing instruments will turn off automatically. You can charge both hearing instruments at the same time.
- ▶ Your hearing instruments turn on automatically and play a start-up melody when removed from the charger.



# Li-ion hearing instruments charging status



## LED Charging status

---



Single orange LED: Charging status 0 - 33 %

---



Two orange LEDs: Charging status 34 % - 66 %

---



Three orange LEDs: Charging status 67 % - 99 %

---



Three green LEDs: Hearing instrument is fully charged.

---